HYGISON®

REF HS0501A

FFP2 Maske / FFP2 Mask

Persönliche Schutzmaske

Personal Protective Mask

CE 2797

EN 149: 2001 + A1: 2009

HYGISON®

REF **HS0501A**
2er Packung

FFP2-MASKE

PERSÖNLICHE SCHUTZMASKE EN149:2001+A1:2009

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suitable for protection against powder, PM2.5 smog, particulates, flu, bacteria, saliva, dust etc. The face mask is good for wind proof and protection the face against cold weather.

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Open the mask and place the bridge of the nose on it. After wearing the mask, press the bridge of the nose to close it. The protective effect is better.

CAUTIONS:

1. Keep the face mask clean before use, do not touch the inner side of facemask.
2. Ensure the face mask fit to face, by adjusting the nosepiece, keeping mask edge close to face.
3. Check the air sealing of mask, do not use if the facemask is broken or with hole or leakage.
4. Do not wash the facemask with water, water will destroy filtering ability and protection.
5. Do not steam the facemask, and do not put it in microwaves.
6. Keep the face mask away from vapor, oils, chemicals, paints, liquid, acid, alkaline objects.
7. Discard the face mask when it gets quite dirty or breathing resistance increase obviously

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The storage temperature is -20~38°C, the storage is moderate ≤ 80%, The validity period is 2 year in the dry indoor environment.

FABRIK / MANUFACTORY: Hunan Dreaming Cloud E-Commerce CO., Ltd

ADRESSE / ADDRESS : Block 1, Smart Tech Park, 57# Huangxing Avenue, Changsha Economic and Technological Development Zone, Changsha, Hunan, China

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Product Name: FFP2 personal protective mask (for personal protective only)

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Address

Block 1, Smart-Hub, 57# Huangxing Avenue, Changsha

Economic and Technological Development Zone, Changsha

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STORAGE CONDITIONS: SHOULD STORE IN DRY, CLEAN AND
WELL-VENTILATED ROOM WITH TEMPERATURE IN 20~38°C, and
relative humidity less than 80%. No corrosive gas environment
without condensation. Please do not use if the product gets moldy or
discolored due to storage damage.

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THIS PRODUCT STANDS PERSONAL PROTECTIVE EQUIPMENT DIRECTIVE
(PERSONAL PROTECTIVE ONLY)

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Hunan Dreaming Cloud E-Commerce Co., Ltd